

Milton®

MAXIMUM PROTECTION

Milton® Sterilising Unit

For use with Milton® Sterilising tablets or fluid



● FIELD OF APPLICATION :

This unit is specially designed for cold water sterilising using Milton tablets or fluid. It allows the disinfection of breast feeding implements (breast pump, nipple shields, milk collection shells), baby accessories (bottles, teats, soothers, rattles, spoons and small plastic toys).

The Milton Sterilising Unit can hold up to 6 standard bottles (less if they are wide necked variety).

The Milton Sterilising Unit features a lockable lid and handle for portability.

The unit used with Milton sterilising tablets or Fluid allows disinfection of all your bottles for the day in 15 minutes.

● METHOD OF USE :

1) **Cleaning of the bottles and accessories**

Wash the bottles and teats using a standard detergent ensuring removal of all milk deposits. Rinse with cold water.

2) **Preparation of the solution**

- Fill the Milton® Unit with 5L of cold tap water.
- Add 1 Milton tablet or 2 measures of Milton liquid.(to the line ie 0.6% v/v)

3) **Disinfection**

- Immerse bottles and accessories into the solution ensuring removal of all air bubbles. Place the grill over to keep them submerged. Insert the teats into the holes in the grill. Place the lid on top.
- In 15 mn everything is disinfected. Bottles can then be added throughout the day to be disinfected and removed when ready to be used.
- Wash hands before removing items from the solution.
- It is not necessary to rinse. Prepare the bottle immediately and cover the teat.

● FEATURES :

- Holds up to 6 standard bottles at the same time.
- Suitable for all types of bottles, breast pump parts and accessories.
- Rounded corners for easy cleaning.

Lid with gripping point :



Laboratoire **RivadIS**

Laboratoire RIVADIS
Impasse du petit Rosé, ZI,
79100 LOUZY - France

Tel : 05 49 68 15 15 / Fax : 05 49 66 16 41